



# Manual – Inserts for water electrodes

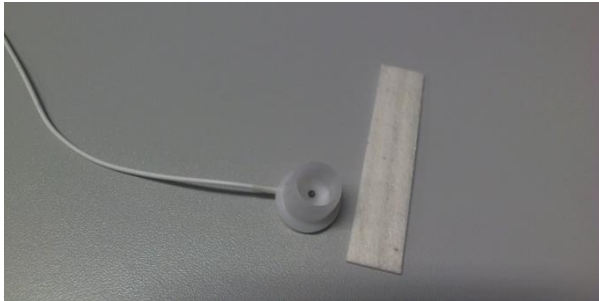

rev.1

Thank you for using the TMSi inserts for water electrodes (81-8480-0002-0) for use with our special water based electrodes (95-7193-3015-0).

This manual describes the step-by-step use of the “inserts for water electrodes” in combination with our water based electrodes.

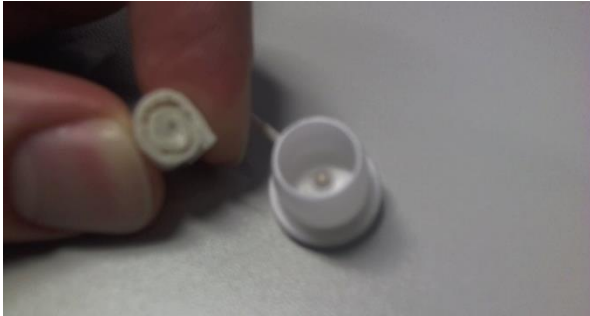


### Step 1

Roll the “insert” like in the photo on the left.



### Step 2

Put the rolled insert in the water-electrode housing.

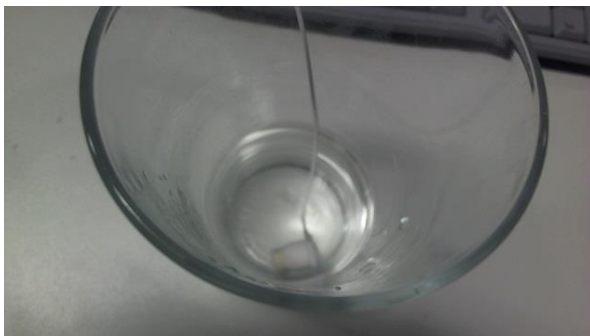

### Step 3

Put the water electrode with the insert into a glass of water.



#### **Step 4**

Here you see an example of a soaked “insert” in the water electrode housing.



#### **Step 5**

Final step. Put the water electrode with the soaked cotton in one of the available fittings of your headcap. The electrode is placed correctly once the subject feels the water on its skin.

If anything remains unclear, please contact us:

TMSi  
Zutphenstraat 57  
7575 EJ Oldenzaal  
the Netherlands  
[support@tmsi.com](mailto:support@tmsi.com)  
☎ +31-541-534603